




Three ways of aView 2 Advance and aCart Compact use:

1. Place the displaying unit on a flat surface of the shelf by folding out the aView 2 Advance stand located on the back side of the device, or
2. Mount the displaying unit on a pole using the Ambu pole mount bracket supplied with the displaying unit (see *Instructions of Use*, section 4.4), or
3. Mount the displaying unit on a standard VESA Mount plate. See the Quick Guide below.



Quick Guide on how to mount aView 2 Advance on the standard VESA plate of aCart Compact

1. Identify the VESA Mount interface on the back side of the aView 2 Advance.

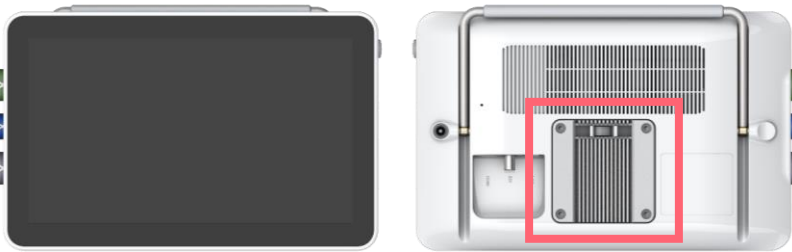


2. Use the flat-head screwdriver to remove the screws. Four screws in total must be removed.




3. Choose four M4 screws of correct length for aCart VESA mounting. Screw length must be 14-16 mm. **IMPORTANT:** Using longer screw lengths will penetrate the lithium-ion battery and result in a fire hazard. Using shorter screw lengths could result in unsecure device fastening.

screw length < 14 mm

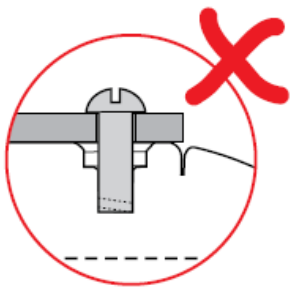

screw length 14–16mm

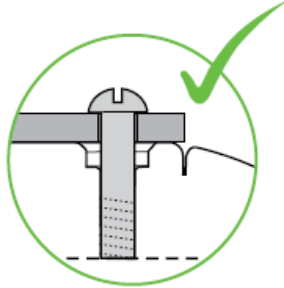

screw length > 16 mm



4. Attach the VESA Mount interface of aView 2 Advance to the VESA Mount plate of aCart Compact by aligning screw holes. Use the screwdriver (Phillips #1) and four screws of 14-16 mm length to secure fastening of the VESA plate.

